

Description

The Loop Pad is for use with hearing aids that have the "T" or "MT" (Loop pickup) facility. The Loop Pad is used with all types of Sarabec Loop Amplifiers, as an alternative to the standard Loop Cable around the room. The pad can be placed either under the user's seat cushion or over the back of an armchair enabling the user to be free of any connecting wires.

The Loop Pad can also be used with other suitable Loop Amplifiers that can accept a 3 Ohm load.


Instructions for use

Place the amplifier in a convenient position near to the television. Place the loop pad in a suitable position on the chair to be used when watching TV. Lay the connecting lead from the Pad to the back of the amplifier. Make sure that the connecting lead is kept out of the way to prevent injury or obstruction.

To connect the cable to the back of the amplifier, see the following diagrams for specific amplifier types.


In all cases, when a terminal is pushed, a hole will appear, push the wire into the hole, making sure that only the wire and not the plastic insulation goes in, then release the terminal.

LA1xx / LA2xx Amplifier Types

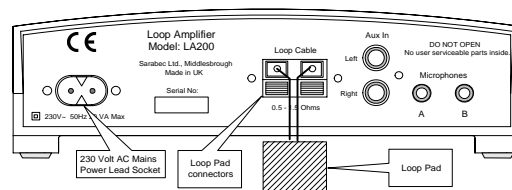

Connect one wire of the lead to the right hand terminal (RED) and the other wire in the lead to the left hand terminal (BLACK).

LA2x / LT1 Amplifier Types

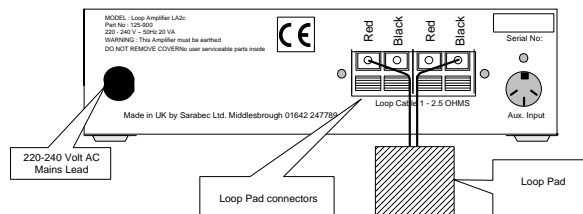

Connect one wire of the lead to the left hand terminal (RED) and the other wire in the lead to the right hand terminal (BLACK). No wires should be placed in the inner pair of terminals.



is a mark of Sarabec Ltd.

Sarabec Ltd 110-053.001 0704 Loop Pad.doc©



Listening Position

When listening to the loop system, the user may find that varying their position in the chair will change the reception. The loop pad should be adjusted for position so that satisfactory sound levels are obtained.

Loop Pad

Middlesbrough TS2 1RH UK
Telephone (01642) 247789
Fax (01642) 230827 Text (01642) 244148