

Description

Your Pocket Vibrating Clock/Timer is supplied for use with a battery.

The Pocket Vibrating Clock/Timer has several novel features:

Alarm Clock


- ◆ choice of 12 or 24 hour display
- ◆ choice of vibrating or audio alarm

Timer

- ◆ triggers the alarm after a pre-set period of time
- ◆ choice of vibrating or audio alarm

Stopwatch

- ◆ stopwatch with pause facility

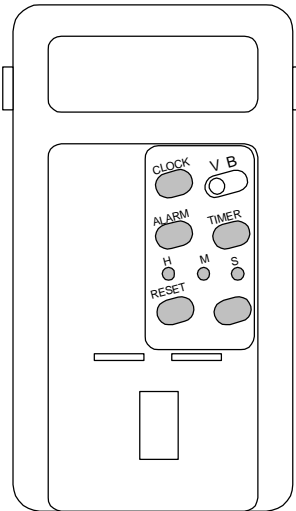

Pocket Vibrating Clock / Timer

Instructions for use

Important

Using a pointed instrument (pen point) or long nails to press the buttons can result in the buttons becoming stuck down. Pressing the button again will normally release it. The top rather than the point of a pencil is normally more suitable for operating the buttons instead of fingers.

Time Setting

To set the time press the CLOCK Button for about 4 seconds and then release. The display will then flash to indicate that the clock is in the time setting mode.

Press the H button to set the hour and the M button to set the minutes. Press and release the H or M button to advance the Hour or Minute by 1. Hold the button down for more than 2 seconds to advance the time quickly.

Pressing the S button will reset the seconds to 00 if the seconds display is showing between 00 and 29. If it is showing between 30 and 59 the seconds will be reset to 00 AND the minutes advanced by one.

To complete the time setting, press the clock button to exit the time setting mode.

12/24 hour clock display

The clock can display the time in either 12 or 24 hour format. To change between the formats press the S/S button for 3 seconds. Whilst the clock is displaying the time in 12 hour format, a "P" will be displayed to indicate PM hours.

Alarm time setting

Pressing the ALARM button will display the alarm time with "AL" between the hours and minutes. To change the alarm time hold the alarm button down whilst pressing the H or M button to set the required alarm time.

To set the alarm to go off at the set time, hold the ALARM button down and press the S/S button. The alarm will appear above the colon.

Slide the V/B switch to select alarm type V = vibrating, B - Audio.

Press any button to stop the alarm.

To turn the alarm function off, hold the ALARM button down and press the S/S button. The alarm symbol will disappear above the colon.

Timer

Press the TIMER button to enter timer mode: the colon does not flash and “TIMER” will be displayed above the seconds on the display.

To set the timer period required use the H, M and S buttons. Pressing and holding the button down for 2 seconds will advance the setting quickly.

To reset the time to 00.00 oo press the RESET button.

To reset only the hours press the H button then the RESET button.

Timer Continued

To reset only the minutes press the M button then the RESET button.

To reset only the seconds press the S button then the RESET button.

To start the countdown timer press the S/S button once. The timer will then count down to 00.00 oo and the alarm will go off. Press any button to turn the alarm off.

To pause the countdown press the S/S button once.

To restart the countdown press the S/S button again.

Stop Watch

Whilst in timer mode (see above) reset the display to 00.00 oo by pressing the RESET button.

Start the stop watch by pressing the S/S button.

Press the S/S button again to stop the watch.

The stopwatch will count to 99.59:59 before starting again at 00.00 oo.

Clock mode can be displayed whilst the timer is in operation and the alarm will operate when either countdown is finished or the set alarm time is reached.

Battery replacement

To replace battery press and slide battery door, remove battery and replace with a new AAA size battery. Make sure the battery is put in the right way as shown and replace the battery door.



Product conforms to the provisions of the following European Union Directives :

- 89/336 & 92/31 Electro Magnetic Compatibility Directives
- 93/68 CE Marking Directive

Pocket Vibrating Clock/Timer