HUMANTECHNIK

**CM-2.4**

**DE**

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Seite 2

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# 1. General information

## 1.1 Introduction

Thank you very much for purchasing the CM-2.4 communication system!

The CM-2 communication system makes it noticeably easier for hearing impaired people to take part in conversations under acoustically difficult situations.

It is for the wireless transmission of speech, music and sound between the CM-2.4 transmitter and the CM-2.4 receiver which may be up to 300 metres apart.

In order to avoid hearing damage due to high volumes, please take care to set the volume to a low level before use.

The manufacturer will not accept any liability for damage and injury in consequence to improper use.

## 1.2 Main features

- Worldwide compatibility due to restriction-free 2.4 GHz transmission
- Large transmission range of up to 300 m
- Fully digitalized crystal clear sound quality
- Up to 20 hours of use per battery charge
- Convenient charging by Micro-USB, compatible with most mobile phone charging adapters

### **1.3 Scope of delivery CM-2.4 Set**

Please check if all of the following components are included:

- CM-2.4 transmitter
- CM-2.4 receiver
- plug-in microphone
- in-ear earphones
- headphones
- neck loop
- 2 belt clips
- 2 neck straps
- 2 USB charging cables
- 2 USB power supplies
- softcase
- user manual
- warranty card

Should any parts be missing, please contact your dealer or the manufacturer directly.

### **1.4 Available accessories**

- |                         |          |
|-------------------------|----------|
| - rechargeable battery  | A-5260-0 |
| - BC-01 belt clip       | A-5291-0 |
| - AM-01 arm band holder | A-5292-0 |

## 2. Quick setup guide

Your *CM-2.4* is already set and ready to use. Adjusting the settings will only be necessary in cases of transmission interferences and / or poor sound quality. Should adjustments be necessary, please note the detailed descriptions from page 20 onwards.

### Setup and use:

Charge the devices: Plug the USB charging cables into the USB jacks of the devices and connect these to regular USB ports (computer, laptop, USB power supplies).

The red charging status LED on the respective device is lit during recharge. The LED switches off when the battery is fully loaded. The charging time is approx. 2 hours.

Plug the microphone into the MIC-input of the transmitter (symbol  on transmitter).

Plug the earphones, headphones or the neck loop into one of the audio outputs EAR1 or EAR2 of the receiver (symbol  on receiver).

If using the neck loop:

Put the neckloop around your neck and set your hearing aid to "T" or "MT" in order to transmit sound information directly to the T-coil in your hearing aid.

Switch the devices on:

Push the ON/OFF switch into the upper position.


Place the microphone close to your conversation partner or other sound source to achieve best results.

Adjust the receiver volume according to your preference:

The volume can be adjusted through levels 00 (Mute) to 09 (maximum volume).

Press the MODE button. The VOL symbol is indicated on the display. Use the »▼« and »▲« buttons to adjust the volume as desired .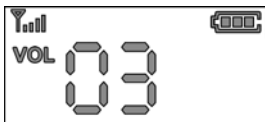

After use:



Switch the devices off: Slide the ON/OFF switch back to the lower position.

Please note:



Please do not connect / disconnect the microphone, headphones or neck loop while the devices are switched on!

## 3. Controls and displays

### 3.1 CM-2.4 transmitter

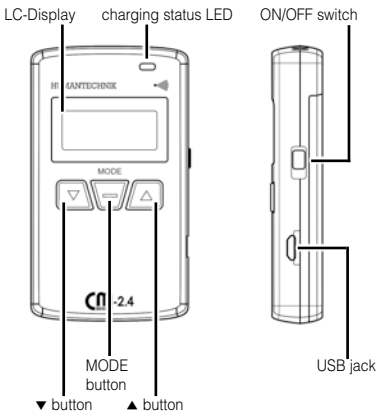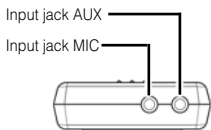## 3.2 CM-2.4 receiver
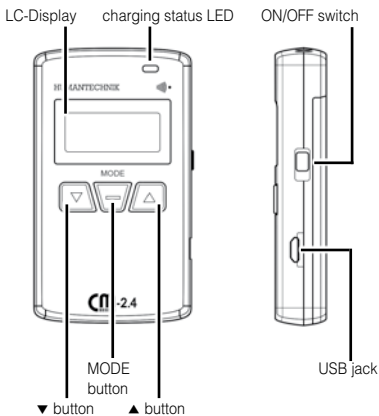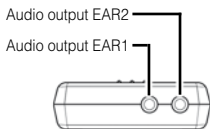
## 4. Detailed description


### 4.1 Power ON / OFF

Slide the ON/OFF switch upwards to switch the devices on. The LC-display will light up and the currently selected group number is shown.

To switch the devices off after use: Slide the ON/OFF switch back to the lower position.

### 4.2 Transmitting and receiving

Transmission will start as soon as the devices are switched on. The symbols , »MASTER« und »TALK« are indicated on the transmitter

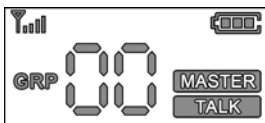The symbol  indicates the signal strength.

### 4.3 Receiver: Volume up / down

The volume can be adjusted through levels 00 (Mute) to 09 (maximum volume).

Press the MODE button. The VOL symbol is indicated on the display. Use the »▼« and »▲« buttons to adjust the volume as desired .

#### **4.4 Receiver: Audio outputs EAR1 and EAR2**

The included earphones or the neckloop can be plugged into any of the outputs EAR1 or EAR2.

The 3,5 mm audio jack makes it possible to connect various audio reproduction equipment, such as earphones, headphones, neckloops but also audio amplifiers and more.

A simultaneous use of both outputs is possible but the volume cannot be adjusted separately.

#### **4.5 Transmitter: Adjusting input sensitivity**

The sensitivity of both the inputs »MIC« und »AUX« can be set from 00 to 05. To set the sensitivity, press the MODE-button. The »VOL« symbol is indicated on the display. Use the »▼« and »▲« buttons to set the input level as desired.

## 4.6 Power supply / recharge

The devices are equipped with a rechargeable lithium-ion battery of 3,7 V / 1100 mAh.

To charge the batteries, plug the USB charging cable into the USB jack of the respective device and connect it to a regular USB port (computer, laptop, USB power supply).

The red charging status LED is lit during recharge. The LED switches off when the battery is fully loaded.

## 4.7 Group number selection

In the rare case of interference, changing the group number can help. The following adjustment has to be made on both the transmitter and the receiver:

Switch the device off. Then press and hold both the »▼« and »▲« buttons while switching the device on.

The symbol „GRP« in the display will blink. Select a group number of your choice between 00 and 39 by pressing the »▼« and »▲« buttons. After the selection, wait for 5 seconds to set the chosen group number. The »GRP«-symbol will stop blinking and the selected group number is displayed.

### Please note:

There should only be one active transmitter within a group! Additional activated transmitters with the same group number will cause interference in the sound quality. Transmitter and receiver must be set to the same group number!

## 4.8 Specifications **CM 2.4**

Dimensions	85 x 50 x 15 mm (H x W x D)
Weight	66 g
Frequency type	ISM 2403 - 2480 MHz
Transmission mode	ISM 2.4 GHz FSK modulation with frequency hopping
Transmission power	Maximum 10 mW
RX-sensitivity	-95 dBm
Latency	< 30 ms
Data rate	2 Mbps
Power	LI-ION 3,7 V / 1100 mAh
Use time / charge	20 hours typical
Range	up to 300 m

### **Conformity**

This device satisfies the following EU directives:

- 2011/65/EG RoHS directive
- 2002/96/EG WEEE directive
- 1999/5/EG R&TTE directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the internet at [www.humantechnik.com](http://www.humantechnik.com).

Specifications subject to possible change.

## **Maintenance and care**

All »CM-2.4«-devices do not require any maintenance. If they become dirty, simply wipe them clean with a soft, damp cloth.

Never use spirits, thinners or other organic solvents. Do not expose them to full sunlight.

In addition, they must be protected against excessive heat, moisture and severe mechanical shocks.

Note: These products are not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, above or near the products.

## **Warranty**

All »CM-2.4«-devices are very reliable products. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by unauthorised persons (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product serial number in any event.**

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, our communal disposal company or your local dealer.

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**Sarabec Ltd**