

Crescendo 10/1 Plus

WARNING : The sound output levels from this system can be very high. It is recommended that the volume control should be turned to the "OFF" position before changing batteries or plugging or unplugging the headphones or microphone.

Instructions for use

1. Plug the microphone into the socket marked <MIC> and the headset into the socket marked <PHONES>.
2. Attach the microphone to the television speaker grille using one [set](#) of the self-adhesive Velcro pads supplied. Peel off the backing paper and press the pad firmly in place. ~~The Velcro already on the~~ microphone will then fasten to the pad if lightly pressed in place. To remove the microphone, pull it away gently. The microphone may be used on other equipment such as a radio or Hi-Fi.
3. Set the television to the normal listening level for the other members of your household and turn the Crescendo's volume control clockwise until the desired headset volume is reached. Remember to switch off the amplifier when not in use by turning the knob fully anti-clockwise, when a "click" will be felt.

The product should give a long and trouble-free service if the following points are observed.

- Treat the headset and microphone leads carefully, they are quite thin in order to be convenient and comfortable to use.
- DO NOT wrap the leads around the amplifier.
- DO NOT leave the microphone lead trailing across the floor to be tripped over.

Battery Changing

To change the battery in the amplifier, turn it over and whilst pressing down, slide the battery cover in the direction of the arrow. The old battery can now be removed by turning the unit over and tapping gently. Unclip the connector from the battery. Fit a new battery making sure that the wires are tucked out of the way. Slide the battery cover into place.

Battery type : PP3, 6R61, MN1604, 6LF22 or equivalent. We would suggest that batteries are bought from a reputable shop with a reasonable turnover to ensure "fresh" stock. There is little difference in the cost of running the Crescendo with either ordinary or alkaline batteries. The alkaline type will last longer but cost more.

Problems and Cures

Problem	Cure
No Sound	<ul style="list-style-type: none">• Headset & microphone in wrong sockets - change round.• Battery exhausted - fit new battery.
Low or distorted sound	<ul style="list-style-type: none">• Battery almost exhausted -fit new battery.• Headset blocked with wax - clean headset ends or fit new forkends
Buzz or squeal after short time	<ul style="list-style-type: none">• Battery almost exhausted, system will seem OK when first switched on - fit new battery
Crackling or Sound goes on & off	<ul style="list-style-type: none">• Dirty microphone plug - clean plug.• Faulty headset or microphone lead - wiggle lead by plug to find faulty lead, replace lead.

If in doubt please contact your dealer for assistance.

The headset ends can be removed for cleaning and removal of wax. They can be washed in warm soapy water but must be thoroughly dried before replacing.

If the system needs to be returned for repair please send the complete system including the amplifier, headset and microphone.

Please state the nature of the fault and the return address. If the system is still under guarantee please state this and provide proof of purchase.

Accessories and Spares

A full list of accessories and spares with prices is shown in our current product guide, available on request. Please check the current price and quote both the description and part number when ordering.

Part No.	Description
330-001	Standard Microphone with 4m(12ft) BLACK lead and Velcro pads
330-007	Standard Microphone with 4m(12ft) WHITE lead and Velcro pads
330-100	Short lead Microphone with 1m(3ft) BLACK lead
330-009	Communicator Microphone with NO lead, direct plug in.
353-090	Crescendo direct audio lead kit to replace microphone
353-092	Crescendo direct audio lead for SCART socket
353-024	Microphone extension lead 3m(10ft)
353-025	Microphone extension lead 5m(15ft)
330-044	Velcro pads kit, One hook type and three black loop type pads To obtain this kit only, please send two second class stamps
310-111	Stethoscope headset complete
302-016	Stethoscope headset lead
301-007	Stethoscope headset foam pads (pack 10)



Product conforms to the provisions of the following European Union Directives :

- 89/336 & 92/31 Electro Magnetic Compatibility Directives
- 93/68 CE Marking Directive

Sarabec is a mark of Sarabec Ltd.  is a mark of Sarabec Ltd.

© Sarabec Ltd. [-104-301.005 0304 Cres Plus10.doc](#) [104-301.003004 12980700 Cres 10.doc](#)

Sarabec 

Crescendo 10/1 Plus

Personal Sound System

Owner's Manual

Introduction

The Crescendo 10/1 Plus system is an easy to use amplifying system for use without a hearing aid. With one simple volume control the user's volume can be easily varied without affecting others.

Your Crescendo 10/1 Plus is supplied ready to use with a battery, microphone, high efficiency stethoscope headset and self-adhesive Velcro™ pads.

Sarabec Limited 15 High Force Road Middlesbrough TS2 1RH
Telephone (01642) 247789 Fax (01642) 230827 Text (01642) 251310