



HUMANTECHNIK

earis®
MAX
buds

DE

Kurzanleitung

EN

Quick guide

FR

Guide rapide

NL

Beknopte handleiding

ES

Guía rápida

Additional information are available on the Humantechnik website:



Lieferumfang | Package contents | **Contenu de l'emballage** | Leveringsomvang | **Contenido del paquete**

Charging case



Earbuds



Comfort tips




Boom tips



Ear fins




USB-Cable

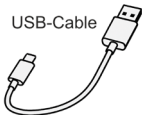


Quick
guide

Safety
guide


EarisMAX Buds laden | Charge earisMAX Buds | Charger earisMAX Buds | earisMAX Buds opla- den | Cargar earisMAX Buds

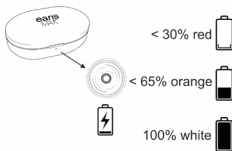

Ohrpasstücke und Ohrfinnen wählen und earisMAX Buds in die Ohren einsetzen | Choose eartips and earfins, then insert the earisMAX Buds into your ears | **Choisissez les embouts et ailettes, puis insérez les earisMAX Buds dans vos oreilles** | Kies oordopjes en oorvinnen en plaats de earisMAX Buds in je oren | **Selecciona las almohadillas y aletas e inserta los earisMAX Buds en los oídos**

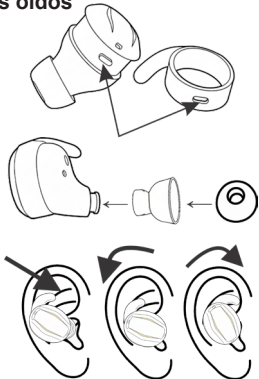

earisMAX Buds mit dem Smartphone per Bluetooth verbinden | Pair the earisMAX Buds with your smartphone using Bluetooth | **Associez les earisMAX Buds à votre smartphone via Bluetooth** | Koppel de earisMAX Buds met je smartphone via Bluetooth | **Emparejar los earisMAX Buds con tu smartphone mediante Bluetooth**

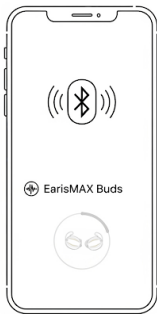

**earisMAX-App herunterladen für individuelle
Einstellungen der Ohrhörer** | Download the ea-
risMAX app to personalize the settings of your ear-
buds | **Téléchargez l'application earisMAX pour
personnaliser les réglages de vos écouteurs** |
Download de earisMAX-app om de instellingen van
je oordopjes aan te passen | **Descarga la aplica-
ción earisMAX para personalizar los ajustes de
tus auriculares**

App



Touch-Bedienung | Touch control | Contrôle tactile | Touchbediening | Control táctil

	 left	 right
 Tap 1x	 - Volume down	 + Volume up
 Tap 2x	 Voice Assistant	 Play / Pause
 Tap 3x	 Previous track	 Next track
 Hold 1.5s	   OFF / HA / ANC	   OFF / HA / ANC



HUMANTECHNIK

earis®
MAX
buds

DA

Hurtigguide

PL

Krótki przewodnik

IT

Guida rapida

FI

Pikaopas

SV

Snabbguide

Additional information are available on the Humantechnik website:



Leveringsomfang | Zawartość opakowania |
Contenuto della confezione | Pakkauksen sisältö
| Leveransomfattning

Charging case



Earbuds



Comfort tips




Boom tips



Ear fins




USB-Cable

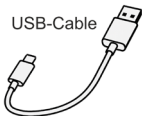


Quick
guide

Safety
guide


Opladning af earisMAX Buds | Ładowanie słuchawek earisMAX Buds | Ricarica degli earis-MAX Buds | earisMAX Buds -kuulokkeiden lataus | Laddning av earisMAX Buds

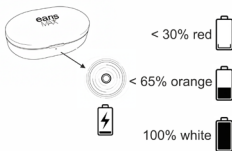

Vælg ørepropper og ørefinner, og indsæt earisMAX Buds i ørerne | Wybierz nakładki i pletwy, włóż earisMAX Buds | **Seleziona gommini e pinne, inserisci earisMAX Buds** | Valitse korvasovit-teet ja siivekkeet ja aseta earisMAX Buds korviin | **Välj öronproppar och öronfenor och sätt in earisMAX Buds i öronen**

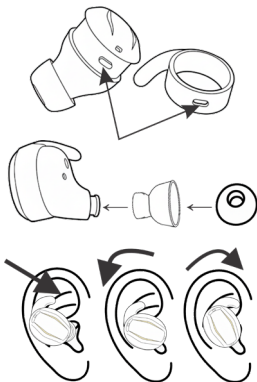

Forbind earisMAX Buds til smartphone via Bluetooth | Sparuj słuchawki earisMAX Buds ze smartfonem przez Bluetooth | **Abbina gli earisMAX Buds al tuo smartphone tramite Bluetooth** | Parita earisMAX Buds älypuhelimesi kanssa Bluetoothin kautta| **Para ihop earisMAX Buds med din smartphone via Bluetooth**

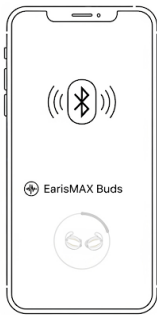

Download earisMAX-appen for at tilpasse indstillingerne for dine øretelefoner | Pobierz aplikację earisMAX, aby dostosować ustawienia swoich słuchawek | Scarica l'app earisMAX per personalizzare le impostazioni degli auricolari | Lataa earisMAX-sovellus mukauttaaksesi kuulokkeiden asetukset | Ladda ner earisMAX-appen för att anpassa inställningarna för dina hörlurar

App



Touch-betjening | Obsługa dotykowa | Controllo touch | Kosketustoiminto | Touch-styrning

	 left	 right
 Tap 1x	 - Volume down	 + Volume up
 Tap 2x	 Voice Assistant	 Play / Pause
 Tap 3x	 Previous track	 Next track
 Hold 1.5s	   OFF / HA / ANC	   OFF / HA / ANC

Technische Daten | Technical specifications | Spécifications | Specificaties | Especificaciones | Specifikationer | Specyfikacje | Dati tecnici | Tekniset tiedot | Tekniska data

Frequency Range:	200Hz - 8kHz
Max. Sound Pressure Level (at 1 kHz):	113dB
Acoustic THD (at 1kHz):	<1%
Bandwith (D/A)	12.5 kHz
Multi-band WDRC	12 Bands
Bluetooth	Bluetooth 5.3
Bluetooth Profile	SBC, AAC, LC3, LE Audio Auracast
Operating Time per Battery Charge:	12 hours / 36 hours with charging case
Charging Time Buds/Case:	both approximately 1.5 hours
Operating Temperature:	0°C to 40°C
Weight incl. Case	50g



These devices comply to the requirements of the following EC directives:

- 2011/ 65/EU + 2015/863/EU RoHS-directive
- 2012/19/EU WEEE-directive
- 2014/53/EU RED-directive

Conformity with the above-mentioned directives is confirmed by the CE mark on the device. CE declarations of conformity are available on the Internet at www.humantechnik.com/service.



Humantechnik's UK authorised representative:

Sarabec Ltd, 15 High Force Road, MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.

UK declaration of conformity available from: Sarabec Ltd.



Cet appareil est conforme aux exigences des directives de IC suivantes :
RSS247 / ICES-003



HUMANTECHNIK

Model No: HMR-01m seriesMAX microphone / receiver

We, Humantechnik GmbH, Im Wörth 25, DE-Weil am Rhein, Tel. +49 (0) 76 21 9 56 89-0,
declare the above comply with the requirements of Federal Communications Commission.
This device complies with Part 15 of FCC rules. Operation is subjected to the following conditions:
1) This device may not cause harmful interferences, and
2) This device must accept any interferences received that may cause undesired operation.

Responsible Party: Daniel Müller

Technical changes reserved.

Humantechnik Service-Partner

D

Germany

Humantechnik GmbH
Im Wörth 25
D-79576 Weil am Rhein

Tel.: +49 (0) 76 21/ 9 56 89-0
Fax: +49 (0) 76 21/ 9 56 89-70
E-mail: info@humantechnik.com

CH

Switzerland

Humantechnik GHL AG
Rastatterstrasse 9
CH-4057 Basel

Tel.: +41 (0) 61/ 6 93 22 60
Fax: +41 (0) 61/ 6 93 22 61
E-mail: info@humantechnik.com

F/B

France
Belgium

SMS
Audio Electronique Sàrl
1 rue des Entrepreneurs
F-68300 Saint-Louis

Tel.: +33 (0) 3 89/ 44 14 00
Fax: +33 (0) 3 89/ 44 62 13
E-mail: sms@audiofr.com

NL

Netherlands

Hoorexpert BV
Gildenstraat 30
NL-4143 HS Leerdam

Tel.: +31 (0) 3 45/ 63 23 93
Fax: +31 (0) 3 45/ 63 29 19
E-mail: info@hoorexpert.nl

GB

Great Britain

Sarabec Ltd
15 High Force Road
GB-Middlesbrough TS2 1RH

Tel.: +44 (0) 16 42/ 24 77 89
Fax: +44 (0) 16 42/ 23 08 27
E-mail: inquiries@sarabec.co.uk

For other service-partners
in Europe please contact:

Humantechnik Germany
Tel.: +49 (0) 76 21/ 9 56 89-0
Fax: +49 (0) 76 21/ 9 56 89-70
Internet: www.humantechnik.com
e-mail: info@humantechnik.com



HUMANTECHNIK

RM414400 · v2025-09-02