


geemarc Wake in Spake Voyager



USER GUIDE

!!! Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of !!!

DESCRIPTION OF THE SAMPLE

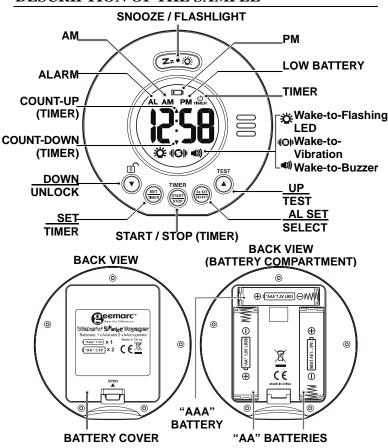

BATTERY INSTALLATION

Your clock requires 1 x AAA and 2 x AA alkaline batteries (included) to operate.

- 1. Place your unit face down on a flat surface.
- 2. Press the latch and remove the battery cover at the back of your unit.
- 3.Insert 1 x AAA and 2 x AA new alkaline batteries into the battery compartment by observing the correct "+" and "- "polarity.
- 4. Replace the battery cover.

AUTO KEY-LOCK

Auto key-lock applies to all other buttons (1 minute after no press of any button). Press and hold **DOWN** button (most right sided button) to release key lock, two soft and short beep tones will be heard.

TO SET TIME AND THE SNOOZE DURATION

- 1. Press and hold **SET / TIMER**, the hour digits "0" flash. Press ▲ or ▼ to set the time in hours (hold ▲ or ▼ to advance rapidly).
- 2.Press **SET / TIMER** again, the minute digits "**00**" flash. Press ▲ or ▼ to set the time in minutes. (hold ▲ or ▼ to advance rapidly).
- 3.Press **SET / TIMER** again, the display shows "**24Hr**" and flashes. Press ▲ or ▼ to select "**12Hr**" or "**24Hr**" time format. There has **AM** icon to indicate the morning time and **PM** icon to indicate the afternoon time.
- 4.Press SET / TIMER again, the display shows "05" and flashes. Press or ▼ to select your snooze time from 5 to 60 minutes.
- Press SET / TIMER again or if no key pressed for approx. 10 seconds to exit the setting mode.

TO SET THE ALARM (default at 6:00 AM)

- 1.Press and hold AL SET / SELECT, the AL appear and the alarm's hour digits' flash. Press ▲ or ▼ to set the alarm in hours. (hold ▲ or ▼ to advance rapidly).
- 2.Press AL SET / SELECT once again, the alarm's minute digits' flash.

 Press ▲ or ▼ to set the alarm in minutes. (hold ▲ or ▼ to advance rapidly).
- 4.Press AL SET / SELECT again, (Sound at HIGH level), press or to select "LO" (Sound at Low level) if need.
- 5.Press AL SET / SELECT once again or no button around 10 seconds to exit the alarm setting mode. Admist alarm setting, press SET / TIMER once to exit and return to nomral time display mode.

TO SELECT THE ALARM MODE

At normal time display mode, press **AL SET / SELECT** one at a time to activate the alarm(s), the corresponding alarm indicator(s) on the right side of the time display will be on:

■ >>)	: only sound alarm is activated
491	: only shaker alarm is activated
*	: only strobe light alarm is activated
 ⊝))	: both sound and vibration alarm are activated
÷Ö: ●))	: both strobe light and sound alarm are activated

TO STOP AND RESET THE ALARM TO COME ON THE NEXT DAY

When alarm is sounding, the corresponding alarm indicator flashes. Press **AL SET / SELECT** once to stop the alarm and reset it to come on the following day. After that, the corresponding alarm indicator stays on the display.

TO USE SNOOZE

When alarm is sounding, presses **Zz** • 🌣 once, the alarm will be stopped temporarily and the corresponding alarm indicator keeps flashing. Alarm will come on again after the set snooze duration is over.

TO USE BACKLIGHT

Press **Zz**• once to active backlight 5 seconds.

TO USE FLASHLIGHT


Press and hold **Zz** • ★ to active flashlight. Press and hold **Zz** • ★ to de-active the flashlight.

TO USE COUNTDOWN TIMER (HOUR-MINUTE TIMER)

- 1.At normal time display, press SET / TIMER once, display change to timer mode, and appear with "0:00" flash.
- Press ▼ to adjust countdown timer in hours, press ▲ to adjust countdown time in minutes.
- 3.Press START / STOP once to start countdown in minutes, then ":" and "▼" flash.
- 4.During counting down, press START / STOP to pause counting down and whole display flashes. Press and hold START / STOP to stop or reset the counting down timer.

- 5.Press START / STOP again to continuous counting down. ":" and "▼ flash again.
- 6.When the timer reaches "0:00" (00 seconds), the clock will vibrates / sounds / flashes with LED and it starts to count up to show the overrun time. Press START / STOP once to stop the timer alarm.
- 7.Press **START / STOP** again, the display shows last count down set time and flashes. Repeat Step 2-5 to set and use countdown timer.

Remark: When timer is active and counting down or up, press the **SET/ TIMER** can return to the time display. The timer icon the normal time display. Will flash on the normal time display.

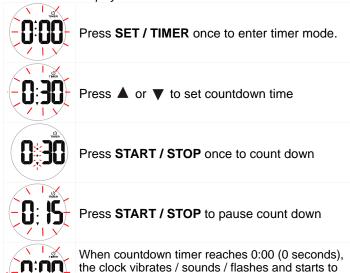

TO USE COUNT-UP TIMER (MAX. 23HR 59MIN.)

STOP once to stop the timer alarm.

count up to show the elapsed time. Press START /

- 1.At normal time display, press SET / TIMER once, display changes to timer mode and flashes. Press and hold START / STOP to reset timer value to" 0:00".
- 2.Press START / STOP once to start count up timer. ":" and "▲" flash.
- 3.During counting, press START / STOP once to pause counting up and whole display flashes. Press and hold START / STOP to stop or reset the counting up timer.
- 4. Press START / STOP once again to continue counting up.



When timer value is reset to **0:00** (Press and hold **START / STOP**), press **START / STOP** once to start count-up.

Remark: When timer is active and counting down or up, press the **SET/TIMER** can return to the time display. The timer icon the normal time display.

TEST MODE

Press and hold **A/TEST** button, strobe light flashing, shaker vibrating and buzzer sounding for 5 seconds and then stops.

LOW BATTERY ALERT

When the battery level becomes low, an low battery icon will apear on the upper middle of the LCD. Repalce batteries with new alkaline batteries.

Remark: frequent use of the vibration alarm will quickly shorten the battery life.

TROUBLE SHOOTING

If your clock displays irrelevant time or does not function properly, which may be caused by electro-static discharge or other interference, open the battery cover, take out the batteries and reinstall them by observing the correct +/- polarity. Your clock will be reset to default settings and you need to set it again.

Low battery level will decrease the vibration and the alarm sound. Once this happens, replace batteries with new ones.

CARE OF YOUR PRODUCT

 Place your clock on a stable surface, away from sources of direct sunlight or excessive heat or moisture.

- Protect your furniture when placing your units on a natural wood and lacquered finish by using a cloth or protective material between it and the furniture.
- 3.Clean your unit with a soft cloth moistened only with mild soap and water. Stronger agents such as Benzene, thinner or similar materials can damage the surface of the unit. Make sure the unit is unplugged before cleaning.
- 4.If the unit is not to be used for a prolonged period, such as a month or longer, remove the batteries to prevent possible corrosion. Should the battery compartment become corroded or dirty, clean the compartment thoroughly and replace the batteries.

SPECIFICATIONS

Alarm duration	: 1 minute		
Snooze duration Adjustable	: 5 to 60 minutes		
Default settings	Time format	24 hour	
	Time	0:00	
	Alarm	6:00	
	Shaker alarm level	Hi	
	Snooze duration	05 (5 minutes)	
	Count-up / countdown time	23 hours 59 minutes	
Audible alarm	: up to 80 dB at 1m		

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at **www.geemarc.com**.

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

CE Directives

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive. The CE declaration of conformity may be consulted at www.geemarc.com

Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.
 Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.





For product support and help visit our website at www.geemarc.com telephone 01707 387602 or fax 01707 832529