

ENGLISH

shake awake®



OPERATING INSTRUCTIONS

KEY FUNCTIONS:

- | | | | |
|---|---------------------------------|---|-----------------------------------|
|  | Selects Vibrating Alarm |  | Snooze Alarm |
|  | Disable Alarm |  | Lock (disables all "Set" buttons) |
|  | Selects Audible Alarm |  | Set Time |
|  | Stop Alarm + Illuminate Display |  | Set Alarm |

1. TO SET TIME

Slide SET switch to "🕒". Press HOUR/MINUTE to actual time. If your **shake awake®** has an AM/PM display, "PM" indicator will appear when afternoon hours are in effect. After setting slide switch to "🔒" (LOCK).

2. TO SET ALARM

Slide SET switch to "🔔" (AL) appears on display.
Press HOUR/MINUTE continuously until desired alarm time appears.
After setting slide switch to "🔒" (LOCK).

3. TO ACTIVATE ALARM

Choose VIBRATING "🔊" or AUDIBLE "🔊" alarm. "🔊" will appear on display.
When alarm activates slide ALARM switch to "🕒" to disable alarm or press "🔔" to stop alarm.
("🔊" will still appear and alarm will repeat every 24 hours.)

4. TO ACTIVATE ALARM WITH SNOOZE FUNCTION (ZZZ)

Slide SET switch to "🔔".
Press HOUR/MINUTE continuously until desired alarm time appears.
Slide SET switch to "ZZZ" position. (ZZZ) will appear on display.
Choose VIBRATING "🔊" or AUDIBLE alarm "🔊".
When SNOOZE alarm activates press "🔔" to stop.
"ZZZ" will flash continuously and alarm will repeat every 8 minutes.
To disable SNOOZE alarm return SET switch to "🔒" position. (ZZZ) will disappear from display.

(Note: To have SNOOZE alarm repeat every 8 minutes you must disable alarm within 1 minute from start.
Flashing (ZZZ) on the display indicates SNOOZE alarm is still activated.)

REMINDER:

shake awake® requires 1 AAA (included) and 2 x AA (not included) batteries.
Replace AA batteries when vibrating motor becomes weak.
When display becomes dim replace AAA battery.

NOTE: Jack on the right side is an auxiliary output jack for an optional **shake awake®** strobe light.

There is a separate low battery warning indicator for the clock and vibration batteries.

The battery sensors check every 45 minutes.

Please insert the 2 x AA batteries first, then insert the 1 x AAA battery, otherwise it will take 45 minutes for the low battery indicator to go out.